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INSTRUCTIONS FOR APPLICATION

- (1) First attach the lead wire to the stimulator and on the other side to two garment electrodes or to one textile and one self adhesive electrode.
- (2) Wash off any lotions, dirt etc. from application site, remove jewelry from area that will be covered by garment and fit to the area of the body to be treated. Ensure the garment fits securely and smoothly, leaving no creases or wrinkles in the garment. Avoid stretching the garment too much.
- (3) If using only one garment, it is necessary to use a reference electrode. To complete the circuit it is recommended to use a self adhesive electrode that has a minimum size of 50x50 mm. (It is important that you do not use smaller sized electrodes, as this may cause adverse reaction during the treatment). This electrode should be placed in close proximity to the garment but must not touch the garment.
- (4) Make sure that the stimulator is turned off prior to wearing or removing the garment.

WARNING: Read instructions before use!

WARNING

- The garment is not sterile.
- Do not apply to broken skin. Should skin rash occur, discontinue use and contact your physician.
- Only connect to battery operated stimulators. Never connect to high frequency surgical device.
- Do not stimulate while driving a motorized vehicle, or operating machinery.
- It is absolutely necessary to consider the contra-indications of the stimulator (read stimulator instruction manual or ask your physician for further details).
- The garment must not be used if torn or damaged.
- The stimulator and connecting lead wires used with this garment must be in compliance with the related standards.
- Do not remove the lead wires by pulling the lead wires themselves!

CLEANING OF THE GARMENT

- Before cleaning the garment take off the snap adapter.
- Wash the garment by hand with warm water (maximum 30°C) and a detergent.
- Alcoholic solutions, bleachers and fabric softeners are not suitable for cleaning the garment.
- Do not iron the garment!
- The garment cannot be sterilized.

NOTES: Always return the garment to the plastic/poly bag provided, after use. Do not leave in humid conditions and store in a dry safe area. The life of the product is a direct result of the use, care and storage of the item.

tic Medizintechnik GmbH & Co. KG

Endelner Feld 9 · D - 46286 Dorsten · Fon: +49 (0) 23 69 · 20 85 - 0
eMail: info@ticmed.de · Web: www.ticmed.de · Shop: www.tic-direkt.de